

# Microchip Guide

BRAND	CHIP # FORMAT	MANUFACTURER	#DIGITS	FREQUENCY	CONTACT
<b>Datamars / PetLink</b>	981xxxxxxxxxxxx	Datamars	15-digit (numeric)	134.2 kHz	1-877-738-5465
<b>HomeAgain</b>	985xxxxxxxxxxxx 4xxxxxxxx	Destron Fearing Digital Angel	15-digit (numeric) 10-digit (alphanumeric)	134.2 kHz 125 kHz	1-888-466-3242
<b>AVID</b>	977xxxxxxxxxxxx xxx*xxx*xxx xxxxxxxx	AVID	15-digit (numeric) 9-digit (numeric) 10-digit (alphanumeric)	134.2 kHz 125 kHz 125 kHz	1-800-336-2843
<b>24 PetWatch</b>	982xxxxxxxxxxxx 0A1xxxxxx	Allflex	15-digit (numeric) 10-digit (alphanumeric)	134.2 kHz 125 kHz	1-866-597-2424
<b>AKC Reunite</b>	956xxxxxxxxxxxx 00xxxxxxxx	Trovan	15-digit (numeric) 10-digit (alphanumeric)	134.2 kHz 125 kHz	1-800-252-7894
<b>Banfield</b>	9810100xxxxxxxx 0D0Dxxxxxx	Datamars Unknown	15-digit (numeric) 10-digit (alphanumeric)	134.2 kHz 125 kHz	1-877-567-8738

## Microchip Lookup Tools:

### PetLink

[www.petlink.net](http://www.petlink.net)

24/7/365 Pet Registry and Recovery Service

**American Animal Hospital Association (AAHA)**

[www.petmicrochiplookup.org](http://www.petmicrochiplookup.org)

Searches most databases in the USA

### PetMaxx

[www.petmaxx.com](http://www.petmaxx.com)

International database that searches more than 31

international pet recovery databases



**PetLink™**

a DATAMARS brand

# How to Use PetLink Microchip Scanners

## Keys to effective scanning

- ✓ Hold the scanner close to or touching the animal. Scan slowly and repeatedly over the entire body to check if a microchip is present. Scanning slowly allows the scanner to cycle through multiple frequencies.
- ✓ In a sweeping motion, scan the pet horizontally in an “S” pattern, starting from the neck to the tail, then vertically up and down extremities.
- ✓ Consider scanning each animal more than once to ensure any present microchip can be localized.
- ✓ Remove collar before scanning. Computers, metal tables, and even fluorescent lighting can interfere with scanning.

## Start scanning at the recommended implantation sites

### Canine & Feline

Implantation should be subcutaneous on the dorsal midline, just cranial to the shoulder blades and scapula.

### Equine

Implantation should be within the nuchal ligament in its middle third, below the base of the mane, or at the halfway point between the ears and withers on the left side.

### Other Species

Microchips can also be implanted in primates, rabbits, birds, reptiles, and other exotic animals. The microchip should be implanted on the animals left side so that placement is consistent with other companion animal species.

Contact us at  
**1-877-PETLINK (738-5465)**

or email us at  
**petlink@petlink.net**

[www.datamars.com](http://www.datamars.com)  
[ussales@datamars.com](mailto:ussales@datamars.com)



**PetLink™**

a DATAMARS brand