

Microchip Guide

BRAND	CHIP # FORMAT	MANUFACTURER	#DIGITS	FREQUENCY	CONTACT
Datamars / PetLink	981xxxxxxxxxxxx	Datamars	15-digit (numeric)	134.2 kHz	1-877-738-5465
HomeAgain	985xxxxxxxxxxxx 4xxxxxxxxxx	Destron Fearing Digital Angel	15-digit (numeric) 10-digit (alphanumeric)	134.2 kHz 125 kHz	1-888-466-3242
AVID	977xxxxxxxxxxxx xxx*xxx*xxx xxxxxxxxxx	AVID	15-digit (numeric) 9-digit (numeric) 10-digit (alphanumeric)	134.2 kHz 125 kHz 125 kHz	1-800-336-2843
24 PetWatch	982xxxxxxxxxxxx 0A1xxxxxxxx	Allflex	15-digit (numeric) 10-digit (alphanumeric)	134.2 kHz 125 kHz	1-866-597-2424
AKC Reunite	956xxxxxxxxxxxx 00xxxxxxxxxx	Trovan	15-digit (numeric) 10-digit (alphanumeric)	134.2 kHz 125 kHz	1-800-252-7894
Banfield	9810100xxxxxxxx 0D0Dxxxxxx	Datamars Unknown	15-digit (numeric) 10-digit (alphanumeric)	134.2 kHz 125 kHz	1-877-567-8738

Microchip Lookup Tools:

PetLink

www.petlink.net

24/7/365 Pet Registry and Recovery Service

American Animal Hospital Association (AAHA)

www.petmicrochiplookup.org

Searches most databases in the USA

PetMaxx

www.petmaxx.com

International database that searches more than 31

international pet recovery databases



PetLink™

a DATAMARS brand

How to Use PetLink Microchip Scanners

Keys to effective scanning

- ✓ Hold the scanner close to or touching the animal. Scan slowly and repeatedly over the entire body to check if a microchip is present. Scanning slowly allows the scanner to cycle through multiple frequencies.
- ✓ In a sweeping motion, scan the pet horizontally in an “S” pattern, starting from the neck to the tail, then vertically up and down extremities.
- ✓ Consider scanning each animal more than once to ensure any present microchip can be localized.
- ✓ Remove collar before scanning. Computers, metal tables, and even fluorescent lighting can interfere with scanning.

Start scanning at the recommended implantation sites

Canine & Feline

Implantation should be subcutaneous on the dorsal midline, just cranial to the shoulder blades and scapula.

Equine

Implantation should be within the nuchal ligament in its middle third, below the base of the mane, or at the halfway point between the ears and withers on the left side.

Other Species

Microchips can also be implanted in primates, rabbits, birds, reptiles, and other exotic animals. The microchip should be implanted on the animals left side so that placement is consistent with other companion animal species.

Contact us at
1-877-PETLINK (738-5465)

or email us at
petlink@petlink.net

www.datamars.com
ussales@datamars.com



PetLink™

a DATAMARS brand

© 2021 Datamars, Inc. All rights reserved. PetLink is a registered trademark of Datamars, Inc.